




PORTABLE COMPRESSOR NEBULIZER

Penetrate into Bronchus & Lung 2-5µm



MICRON NEBULIZATION FINE PARTICLES

MMAD: 3.6 - 4.0 µm

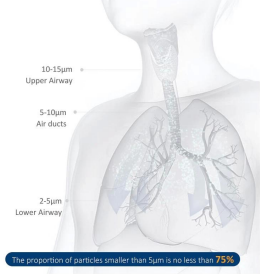


QUIET ENVIRONMENT SILENT ATOMIZATION

Low noise design, easy for children to sleep



USE BASIC STEPS

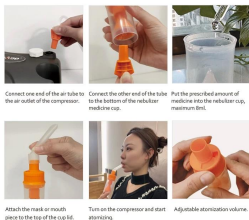

Connect one end of the air tube to the air outlet of the compressor. Connect the other end of the tube to the bottom of the nebulizer. Put the prescribed amount of medicine into the nebulizer cup, maximum 5ml.

Attach the mask or mouthpiece to the top of the cup lid. Turn on the compressor and start atomizing. Adjustable atomization volume.

|||||



|||||

