

Able Assist Adjustable Transfer Aid provides a safe and easy solution for a Carer wishing to transfer a Client.

Able Assist Adjustable Transfer Aid is ideal for users that can raise from a seated to a standing position. When transferring from room to room it enables the user to participate in the transfer which helps improve muscle tone by having their legs supported by the knee brace and they can hold on to the horizontal bar in front of them this also provides the carer with a safe and solution when transferring. The narrow width of the Able Assist enables the carer to manoeuvre through standard doorways with ease.

### Features:

1. One Able Assist Adjustable Transfer Aid
2. Safe and easy solution for transferring patients
3. Spreadable legs for added stability
4. Ideal for those that can raise from a seated to a standing position
5. Manoeuvred through standard doorways with ease
6. Knee brace helps to support legs
7. 2 braked wheels maximise security and stability
8. Can be wheeled directly up to a toilet

<b>Product Name</b>	Able Assist Transfer Aids
<b>Net Weight</b>	33.5kg
<b>Weight Capacity</b>	200kg
<b>Height</b>	35.8" * 25.2" * 4.2"
<b>Width of Base</b>	16.5" - 31.5"
<b>Function</b>	Perfect for room to room transfers
<b>Application</b>	Home care, nursing home or hospital, etc.

### Detailed Description:



Flip up the Seat Pads so the User can occupy the Able Assist[]



The user places his/her feet on the stable anti-slip foot plate, and gets support from the adjustable lower leg support.



You can use the able assist in the bathroom and can be wheeled up to a toilet, the wheels can be positioned at either side of the toilet offering the user more dignity when sitting down.



Fitted with 4 wheels that enhance manoeuvrability and enable users to be transferred with ease, 2 of the wheels have breaks that ensure maximum security and stability when transferring a patient.