

This is a quiet and hard-working compressor nebulizer. It offers higher liter flows, shorter treatment times and lower sound levels than many other economy compressor nebulizers.

**Detailed Description:**

Ultra compact design makes it extremely portable.

Powerful piston pump is efficient dependable and quiet.

Delivers consistent particle size of 4.5 microns and fast treatment times.



Suitable for A Variety of Situations



**PORTABLE COMPRESSOR NEBULIZER**

Penetrate into Bronchus & Lung 2-5μm



**QUIET ENVIRONMENT SILENT ATOMIZATION**

Low noise design, easy for children to sleep



1. Nebulizer machine  
 2. Medicine cup  
 3. Air tube  
 4. Adult mask  
 5. Child mask  
 6. Mouthpiece  
 7. Elastic strap

**USE BASIC STEPS**



Connect one end of the air tube to the air outlet of the compressor.



Connect the other end of the tube to the bottom of the nebulizer medicine cup.



Put the prescribed amount of medicine into the nebulizer cup, maximum 8ml.



Attach the mask or mouth piece to the top of the cup lid.




Turn on the compressor and start atomizing.



Adjustable atomization volume.


More Details

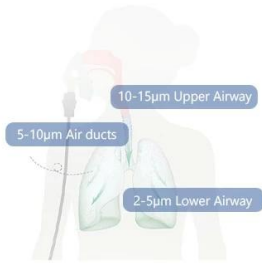

**Micron level nebulization—2µm-5µm**  
Nebulized particles directly reach the upper respiratory tract, trachea, alveoli and lower respiratory tract, making it easier to absorb the medication.



**High atomization efficiency**  
Smooth atomization for continuous operation.

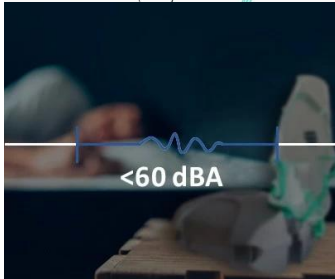

**Low noise design**  
Low power, low vibration, baby atomization does not resist.